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**745/00****PNEUMATIC HAMMER CHISEL FITTED WITH 3 TOOLS****USE**

This chisel has been designed exclusively for deflasking and working stone models in dental technology.

The use of this instrument saves up to 80% of the deflasking time and permits rapid creation of the necessary spaces on the model.

TECHNICAL DATA

Operative pressure: 5-6-atm.

Consumption: 40/50 litres of air per minute.

Weight: 230 grams (300 gr. including packaging).
The pneumatic functioning guarantees long life.

INSTALLATION

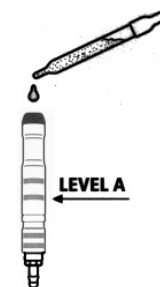
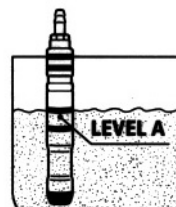
Warning: before connecting the chisel to the pipe, open the air for one minute to blow away any residues of sand eventually deposited; sand or microscopic metallic particles can block and damage severely the motor.

It may occur also that, after opening the air, the piston does not start because it is in the neutral phase: in this case, pushing the tool against the desk will start the cycle again.

MAINTENANCE

Since the compressor produces condensate, the chisel **must be lubricated** (even if the compressor has a filter) with a few drops of Diesel fuel oil at the end of every week: apply the oil to the lubrication point (see Fig.1) and leave the chisel in vertical position.

If the chisel is left idle for more than 10 days it should be immersed without the tool in a in of Diesel oil **up to level A** for the whole period (see Fig.2). It is recommended to keep the tools always sharp.

**WEEKEND****HOLIDAY****REFILLS**

745/20 Small blade tool (7 mm)

745/21 Medium blade tool (9 mm)

745/22 Large blade tool (11 mm)

GUARANTEE

One year (the guarantee is valid only if the chisel is used according to the enclosed instructions of use and maintenance).

Carlo De Giorgi s.r.l. Via Tonale 1 – 20021 Baranzate (MI) – ITALY www.degiorgi.it
Tel. +39 02 3561543 / Fax +39 02 3561808/ e-mail: info@degiorgi.it