carlo de giorgi

Extra Shine

WHITE ACCELERATOR

PROFESSIONAL WHITENING SYSTEM

- ① Istruzioni per il dentista
- ② Information for the dentist
- ③ Información al dentista

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WHITE ACCELERATOR EXTRA SHINE

Introduction

WHITE ACCELERATOR EXTRA SHINE is a dual syringe-system to whiten the teeth in the dental office quickly and easily – normally in a single session.

WHITE ACCELERATOR EXTRA SHINE 37,5 % Hydrogen Peroxide (H_2O_2) recommended for the use with bleaching lamps or laser.

The gel is highly viscous to prevent dripping. It can be applied easily to the teeth using the mixing tips. The mixed gel is red. The used syringe can be re-closed allowing the remaining gel to be used for the next patient, with a new mixing tip.

Warning

Because of the high concentration of Hydrogen Peroxide WHITE ACCELERATOR EXTRA SHINE is highly reactive. To avoid contact with skin or eyes we recommend the use of gloves and safety goggles. If the gel does come into contact with skin, remove it and rinse immediately with water. White spots may appear in the affected area and you may experience a burning sensation. The symptoms should disappear after approx. 30 – 60 minutes. If the gel comes into contact with the eyes, rinse immediately and consult a doctor. Any dental work (crowns, fillings etc.) should be done 1 – 2 weeks after the treatment in order to match the colour of the newly whitened teeth. Please note that the adhesive strength of any dental work may be lessened in the 2 – 3 days following bleaching treatment. Do not perform whitening treatments under anaesthetic.

De Giorgi does not recommend the use of WHITE ACCELERATOR EXTRA SHINE on patients who are pregnant, lactating or under the age of 16.

1) Preparation

- After examining the patient the current tooth colour should be documented. It may be advisable to begin the treatment with a professional teeth cleaning. The teeth should be kept free by using a lip holder, lip retractor or similar instrument. If necessary, use cotton wool balls and apply Vaseline to the patient's lips.

2) Application of WHITE GINGIVAL BARRIER

- Attach the dispensing tip to the top of the syringe of the WHITE GINGIVAL BARRIER.
- Dry the gingiva carefully to ensure that the WHITE GINGIVAL BARRIER will hold and seal during the complete treatment. Place the WHITE GINGIVAL BARRIER directly on the gingiva and cover 3–4 mm of the gingiva and 0.5–1 mm of the teeth. Now light-cure the WHITE GINGIVAL BARRIER with a UV-lamp for 30 – 40 seconds in an arching movement over the material. Instead of the WHITE GINGIVAL BARRIER a rubber dam can be used.

3) Whitening procedure

- Attach the mixing tip to the top of the dual-syringe of the WHITE ACCELERATOR EXTRA SHINE Gel. Apply the gel directly to the surface of the teeth. The gel should be applied about 1 – 2 mm thick (avoid contact with other parts of the body and clothes).
- Allow the gel to work about 12 – 15 minutes. Reduce the application time and cycles if patient experiences sensitivity. WHITE ACCELERATOR EXTRA SHINE can be activated with a LED lamp (POLILUX ACCELERATOR PLUS). Special bleaching lamps or lasers can be used (check with the manufacturers for details about this kind of usage).
- Remove the gel from the teeth using suction – do not rinse – then apply a fresh layer of gel to the teeth. This process can be repeated three times.
- After the last treatment remove the gel first using suction and then by rinsing.
- Now the WHITE GINGIVAL BARRIER, lip holder and cotton wool balls can be removed. Finally the patient should rinse their mouth with warm water.

When bleaching lamp is used:

Place bleaching lamp close to the patient's mouth during the gel application and turn the lamp. In case of the De Giorgio POLILUX ACCELERATOR PLUS only eye protection with safety goggles for patient/operator are mandatory. For other bleaching lamps follow instruction of the lamp manufacturer.

After less than one hour your patient will be able to enjoy his new white smile – thanks to your treatment.

The tooth colour after treatment should also be documented for your records. After the treatment apply a fluoride varnish to the teeth. You can finish off the treatment and lower the risk of unpleasant sensitivity for the patient With WHITE DESENTIZISING MOUSSE – apply to the teeth and leave for about 10 minutes.

Side effects

If the patient feels a burning sensation in the mouth, treatment should be stopped at once and the gel should be removed. Sensitivity can be reduced by using a fluoride gel or WHITE DESENTIZISING MOUSSE after the treatment. For safety reasons, the patient should also wear safety goggles.

Storage and shelf life

WHITE ACCELERATOR EXTRA SHINE must be stored in a cool place or refrigerator (4 – 12 °C). The expiry date on the package or syringe refers only to products that have been stored correctly. Do not freeze the gel and protect it from heat and sunlight.

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