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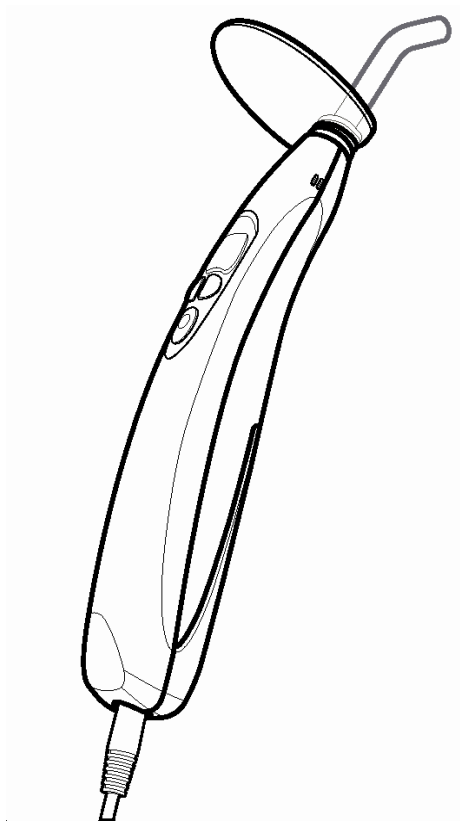
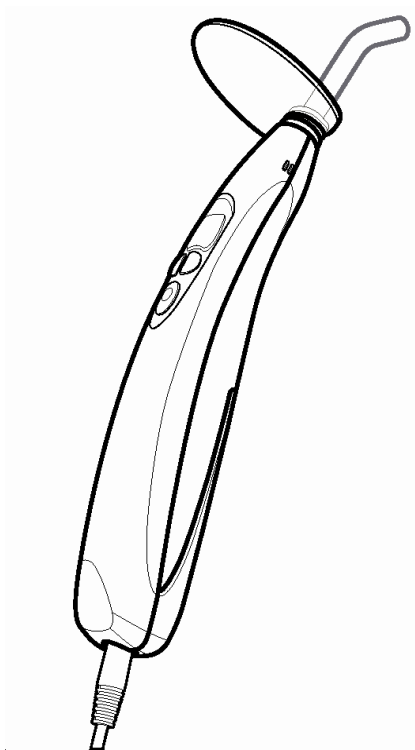
Tel. +39.02.356.15.43 a.s – Fax +39.02.356.18.08

CONTENTS

1 - OPERATING PRINCIPLE AND USAGE	3
2 - STRUCTURE AND COMPONENTS	3
3 - TECHNICAL SPECIFICATIONS	3
4 - INSTALLATION AND DISASSEMBLY	4
5 - OPERATION	4
6 - CAUTIONS	6
7 - CONTRAINDICATIONS	6
8 - DAILY MAINTENANCE	6
9 - TROUBLESHOOTING	8
10 - PACKING LIST	9
11 - AFTER SERVICE	9
12 - STORAGE AND TRANSPORT	9
13 - SYMBOLS	10
14 - ENVIRONMENTAL PROTECTION	11

1 - OPERATING PRINCIPLE AND USAGE

1. The LED.D adopts the principle of light radiation for setting light-sensitive dental composites. Setting is achieved by aiming the curing light at the composite for a short time.
2. This product is designed to set dental composites employed in tooth restoration and whitening procedures.

2 - STRUCTURE AND COMPONENTS

The LED.D (dental) appliance is composed mainly of a high power LED, an optical fibre, a main unit and a charger.

3 - TECHNICAL SPECIFICATIONS

1. Power supply: rechargeable Lithium battery
 - a) Battery voltage and capacity:
 - 3.7V 1400mAH
 - Battery model: ICR18490
 - b) Charger input: AC100~240V 50/60Hz
output: DC5V 1A
Built-in charger fuse: 1A/250V
2. Light source:
 - (1) 3 W high-power blue LED light
 - (2) wavelength: 420nm - 480nm
3. Light intensity: 800mW/cm² (invariable before “low battery” alarm)
4. Working conditions:
 - (1) ambient temperature: 5°C to 40°C
 - (2) relative humidity: ≤ 80%
5. Dimensions: 32 x 38 x 202 mm
6. Net weight: 300g
7. Power consumption: ≤ 5W
8. Degree of protection against electrical shock: Class II
9. Degree of protection against electrical shock: type B
10. Degree of protection against harmful ingress of water or particular matter: ordinary equipment (IPXO)
11. Degree of safety in the presence of flammable anaesthetic mixture with air, oxygen or nitrous oxide: NOT suitable for use in these working conditions.

4 - INSTALLATION AND DISASSEMBLY

1. Remove the red cap from the optical fibre and insert the metal part into the front of the LED.D. (screw to install, making sure that it is straight).
2. Fix the light shield to the top of the optical fibre.
3. To disassemble follow the indications above in reverse order.
4. Battery replacement: press the button on the bottom of the main unit, pull the cover back and open it, take the battery out, then disconnect the plug. Connect the plug of the new battery correctly then fix the battery in place, push the cover back up on the main unit until it is closed correctly.

5 - OPERATION

1. Press the “mode” button to select the required operating mode (the indicator for the mode selected comes on).
 - 1) “” Full power: when the ON/OFF button is pressed the blue light comes on at maximum intensity.
 - 2) “” Ramp: The power of the blue light increases gradually reaching maximum power 5s later.
 - 3) “” Pulse: The blue light works in pulses.
2. Press the “time” button to choose the setting time. 8 working time modes are available: 5s - 10s - 15s - 20s - 25s - 30s - 35s - 40s.
3. When the operating mode and time are both set, the curing light will memorize the settings and work with these settings during the following session.
4. When operating, aim the blue light at the correct position and then press the ON/OFF button, the machine produces a “beep” sound and operates with the blue light in accordance with the selected settings. The digits on the screen commence a countdown and the appliance stops working when the countdown reaches “0”, after which the digits on the screen return to display the selected time setting.
5. During operation the appliance can be stopped at any time by pressing the ON/OFF button.
6. An acoustic signalling device inside the appliance produces a “beep” sound whenever the ON/OFF button is pressed and when the set operating time has terminated.

7. An electrical battery level monitoring device is incorporated in the appliance. When low battery power is detected, the mode indicator light comes on indicating the need to recharge the battery as soon as possible.
8. To recharge the appliance, carefully remove the protective cover from the socket on the bottom of the main unit and insert the output jack of the charger into socket. Remove the jack from the socket when the battery has been recharged and replace the protective socket cover. When the curing light is charging the three mode indicators light up in sequence. When charging is finished all three mode indicators light up simultaneously.
9. During the charging process the appliance can enter standby mode by pressing the button and be switched to normal operating mode and used.
10. When normal operating mode switches back to stand-by the appliance will return to charging status if the battery has not been fully charged, in this case the battery charging indication is on.
11. In the event of a faulty battery or in the absence of a battery, the appliance can still be used by connecting it directly to the charger.
12. When the "low battery" alarm is announced the appliance can still be used but the light intensity will gradually decrease until it eventually enters safety mode: all functions are locked until the appliance is connected to the power supply or the battery is recharged/replaced. The appliance will then return to normal operation.
13. After operation, please clean the fibre with a lint-free cloth in order to guarantee the correct light intensity during the following application.
14. After 2 minutes of idle status, the appliance will automatically switch itself off. To turn the appliance back on, press any button.
15. The effective light intensity of the LED is much higher than that provided by a halogen lamp curing light. The setting depth of the dental composite after exposure to the curing light for 10 seconds is not less than 4mm.
16. The optical fibre can be detached and autoclaved at a temperature of 135°C and pressure of 0.22Mpa.

6 - CAUTIONS

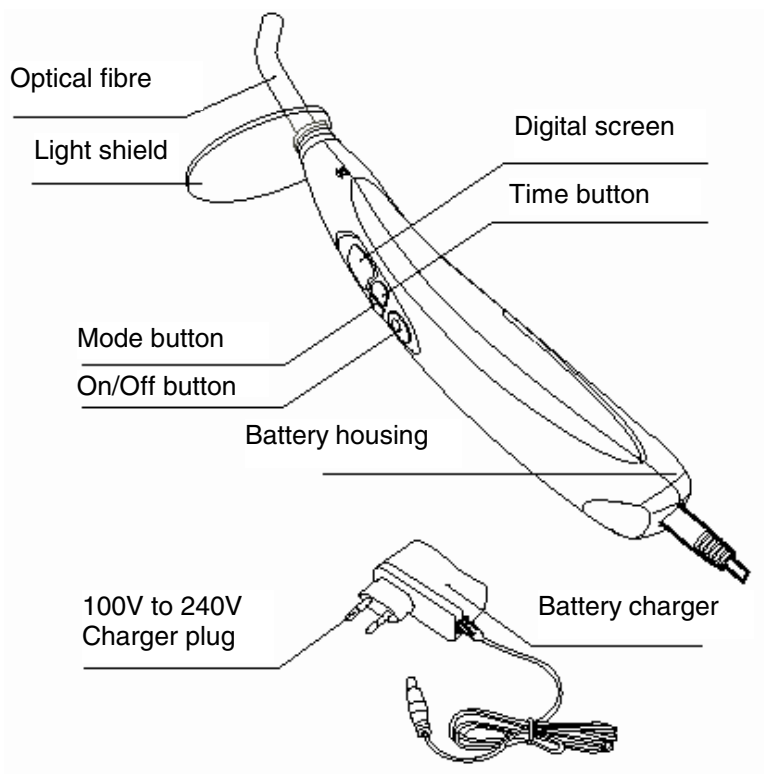
1. During operation, the light should be aimed directly onto the composite to ensure effective setting.
2. This appliance must not be used to treat patients who may manifest an allergic reaction.
3. Avoid aiming the light directly into eyes.
4. Only use the original charger, other chargers may damage the battery or circuit irreparably due to incompatibility.
5. It is forbidden to touch the charging pin of the main unit with metal objects or other conductors as this could damage the circuit board and the Lithium battery.
6. Charge the battery in a cool and ventilated room.
7. To avoid the risk short circuit or electrolyte leakage, do not disassemble the battery.
8. Do not squeeze or shake the battery, do not short circuit the battery and do not store the battery with metal objects.

7 - CONTRAINDICATIONS

1. It is forbidden to use this appliance on patients who may manifest an allergic reaction.
2. The use of this appliance is forbidden if either the doctor or the patient has a pacemaker.
3. The appliance must be used with maximum caution if the patient being treated is a pregnant woman, child or person with heart problems.

8 - DAILY MAINTENANCE

1. Only the optical fibre of this equipment can be autoclaved under high temperature and pressure. Other parts should be cleaned using a cloth dampened with water or neutral sterilized liquid only. Do not soak. To avoid damaging the plastic surface and operator panel, do not clean with volatile liquids or solvents.
2. Carefully clean the optical fibre to remove traces of dental composite from the surface. Failure to do so will affect the effectiveness of setting during subsequent use and the life-span of the optical fibre itself.

9 - TROUBLESHOOTING

Fault	Possible cause	Solution
No indication, no response	1. Battery is drained. 2. Faulty battery.	1. Charge the equipment/ Change battery. 2. Change battery.
“E” appears on the screen	Faulty main unit.	Send to Customer Services for repair.
“E1” appears on the screen	Low battery.	Charge the battery; if E1 appears again within 15 minutes change the battery.
Weak light intensity	1. The optical fibre is not correctly installed. 2. The optical fibre. is cracked. 3. There are composite residues on the tip of the optical fibre.	1. Reinstall the optical fibre correctly. 2. Replace with a new optical fibre. 3. Clean.
The equipment is not charging when the charger is connected	1. The charger is not connected properly. 2. Faulty or incompatible charger.	1. Connect the charger to the power supply. 2. Change the charger
Effective duration of the battery is short	Diminished battery capacity.	Replace with a new battery
The mode indicator twinkles when charging	1. Low voltage. 2. Short-circuit of the battery.	1. Back to normal after 15 minutes of charging 2. Replace with a new battery

10 - PACKING LIST

(The components of the machine are listed in the packing list).

11 - AFTER SERVICE

1. We offer a one year warranty from the date of purchase.
2. The manufacturer is not responsible for any damage caused by unqualified persons or by improper use.
3. The manufacturer reserves the right to modify the design of the appliance, the technical data, the accessories, the Use and Maintenance Manual and the contents of the original packing list at any time and without prior notice. In the event of discrepancies arising between the documentation and the appliance contained in the packaging, the latter is to be considered correct.

12 - STORAGE AND TRANSPORT

1. This appliance must be handled with care. Kept away from vibrations and install or store in a cool, dry, ventilated room away from direct sunlight.
2. Do not keep the appliance near combustible, toxic, caustic or explosive materials.
3. The appliance must be kept in an environment with the following characteristics:
 - relative humidity: $\leq 80\%$
 - atmospheric pressure: 75 kPa - 106kPa
 - temperature: -10°C to $+55^{\circ}\text{C}$.
4. Avoid excessive impacts and shaking during transport. Position the appliance carefully and do not overturn.
5. Do not transport it together with hazardous articles.
6. Protect the appliance from rain, snow and direct sunlight during transport.

13 - SYMBOL INSTRUCTION

Trademark

Type B applied
part

IPX0

Ordinary
equipment

Caution! Consult
accompanying
documents

Class II
equipment

CE mark

FDA certified

Screw in/out

For indoor use only

14 - ENVIRONMENTAL PROTECTION

The crossed out wheelie bin symbol indicates that the product must not be disposed of along with other household waste.

The differentiated disposal of this appliance at the end of its working life is organized and managed by the manufacturer. When the user wishes to throw this appliance away he/she must contact the manufacturer and follow the system adopted by the same to dispose of it correctly.

Suitable differentiated disposal, to enable possible recycling or environmentally compatible elimination of this appliance, helps to limit any undesirable impact on health and the environment and promotes the reuse and/or recycling of the materials which went into making up the appliance.

Sanctions, in accordance with the relative legislation in force in the country of use, will be imposed on the holder if the apparatus is disposed of illegally.

EC DECLARATION OF CONFORMITY

The undersigned:

CARLO DE GIORGI S.r.l.
Via Tonale, 1
20021 BARANZATE (MI) - I

declares under its own responsibility that the appliance:

Type: **Rechargeable LED curing light**
Art. **675/00 LED.D**

Classification in accordance with Directive 93/42 EEC: I

hereby described as:

a rechargeable LED curing light for use in orthodontic surgeries,

conforms to the Legislative Dispositions transposed from Medical
Devices Directive 93/42/EEC

Name: Ariberto De Giorgi
 Managing Director

carlo de giorgi s.r.l.

Via Tonale, 1 – 20021 Baranzate (MI) – Italy
Tel. +39.02.356.15.43 r.a. – Fax +39.02.356.18.08
<http://www.degiorgi.it> E-Mail: info@degiorgi.it