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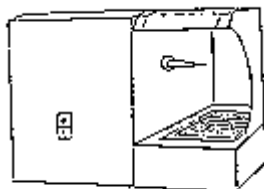
**ASPIRATION HOOD WITH POLISHING LATHE - SENIOR MODEL ART. 350/00
INSTALLATION, USER AND MAINTENANCE MANUAL**

INSTALLATION: The Senior model aspiration hood with polishing lathe will operate in conditions of optimum safety when correctly installed in the laboratory. Position the unit on a level surface on a stable support near a 230 V electrical socket outlet, ensuring it is at least 10 cm from all walls or obstructions to ensure proper ventilation.

ELECTRICAL CONNECTION: the appliance is equipped with an electrical cable fitted with a plug for connection to a 230 V 50 Hz mains socket outlet.

Before making the electrical connection, check that the values of the electrical mains are commensurate with the data shown on the appliance dataplate and that the electrical line is protected by a thermal-magnetic circuit breaker. Check also that the electrical socket outlet is compatible with the plug on the appliance power cable and that the mains circuit is complete with an earth wire.

OPERATION: this appliance is designed for polishing small items in jewellers and watchmakers workshops and dental laboratories, etc.. The appliance must be used by skilled personnel trained in the operation of this type of equipment and with experience in the type of work to be performed. The lathe is equipped with a "taper coupling" screwed onto the motor arbour; before connecting the appliance to the power supply, always check that the coupling is securely attached to the motor arbour.

Fit the polishing brush to the taper coupling and screw it tight, pressing the "START 1" pushbutton. This action switches on the light in the enclosure, starts the lathe and also starts the aspiration system. In the event of a temporary electrical power loss the lathe will stop: when power is restored the lathe will not restart until the "START 1" pushbutton is pressed again. To switch off the appliance press the "STOP O" pushbutton. This action deactivates all functions of the appliance.

CLEANING AND MAINTENANCE: Important: before commencing cleaning or maintenance work unplug the appliance plug from the electrical socket outlet. The only user maintenance operation is cleaning or replacement of the filters, which must be performed on the basis of the intensity of use of the appliance. Proceed as follows: remove the protective grille from the work enclosure in order to gain access to the filters; remove the filters and clean or replace; refit the filters in the filter compartment and then refit the grille. For more thorough cleaning, periodically unscrew the two screws that secure the filter holder basket in order to gain access to and clean the space below the basket.

Do not use water jets, water spray or aerosols to clean the exterior of the appliance. Use exclusively a damp cloth.

To replace the 230V 25W lamp remove the screw securing the polycarbonate diffuser. After changing the lamp refit the diffuser.

During operation of the appliance the noise level is approximately 78 dB(A) – use hearing defenders.

Warning: the manufacturer declines all liability for possible injury to persons or animals or damage to property caused by incorrect or improper use of the appliance. Always wear personal safety accessories, namely gloves, safety glasses and a facemask.

Do not wear rings, bracelets, watches, chains or necklaces, which could become entangled in moving parts of the appliance or which could, in any event, represent a source of danger for the operator, other persons, animals or property.

- Never remove the protective screen
- Do not remove the protective grille when the aspirator is running
- Do not attempt to fit or remove brushes when the lathe is running

Warranty: this appliance is guaranteed against defects of material or workmanship for 12 months from the date of purchase; all defective parts will be replaced free of charge.

The manufacturer's liability is limited to the repair or replacement of parts which, at its sole discretion, are considered to be in improper working order.

The warranty does not cover the replacement of the entire appliance. Any defects not clearly related to the materials utilized or workmanship must be assessed by the manufacturer in the factory.

In the event of unjustified warranty claims, all expenses for the repair of the appliance and/or replacement of parts will be charged to the purchaser. The appliance must be shipped to the manufacturer's factory at the expense of the purchaser; the purchaser will also be charged for shipping costs incurred in returning the appliance. Any appliances sent to the manufacturer "carriage forward" will not be accepted and will be returned to the sender.

The warranty will be invalidated in the following cases:

- if the appliance has been tampered with or repaired by unauthorized persons;
- if the defect is found to have been caused by improper use of the appliance or by the failure to observe the instructions in this user and maintenance manual;

Costs deriving from damage caused during transport or during installation of the appliance, or any other damage not attributable to the responsibility of the manufacturer, will not be covered under the terms of the warranty.

The warranty does not cover appliances that have been repaired or tampered with by unauthorized persons; the warranty also excludes interventions regarding modification of the controls. The warranty shall not be extended further to technical interventions. Costs relating to any extensions in the terms of the warranty specified or promised by the dealer must be borne entirely by this latter.

The warranty takes effect as from the date on the receipt or purchase invoice, a photocopy of which must be forwarded with the appliance in the event of warranty claims. If this document is not submitted, any work performed will be automatically charged for.

The warranty expressly excludes compensation for direct or indirect damages of any whatsoever nature in relation to persons or property such as may be calculated during periods in which the appliance is unavailable for use.

TECHNICAL DATA	
Sound pressure	78 dB(A)
Suction motor power	400 W
Aspiration capacity	400 m ³ /h
Single-phase power supply	230 V 50 Hz
Dimensions	50 x 34 x 33 cm
Weight	19 kg
Total power consumption	500 W

The manufacturer reserves the right to change specifications without notice.