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GUILIN WOODPECKER MEDICAL INSTRUMENT CO., LTD

Registration Add and Manufactory Add:

Information Park, National High Tech Zone. Guilin, Guangxi, P. R. China 541004

Instruction Manual for LED.E (curing light)

carlo de giorgi s.r.l.

Via Tonale, 1 – 20021 Barenzate (MI) – Italy

Tel. +39.02.356.15.43 r.a. – Fax +39.02.356.18.08

Contents

1. Principle and usage	3
2. Structure and components	3
3. Technical specifications	3
4. Installation and disassembly	3
5. Operation	4
6. Cautions 	4
7. Contraindication	5
8. Daily maintenance	5
9. Trouble shooting	5
10. Packing list	6
11. After-sales services	6
12. Storage and transportation	6
13. Symbol instructions	6
14. For technical data, please contact	6
15. Environmental protection	6


1. Principle and usage

- 1) LED.E adopts the principle of ray radiation to solidify the light-sensitive resin by shooting at it in a short time.
- 2) This product is used for dentistry. It has the function of accelerating dental restoration and solidifying the material of dental whitening.

2. Structure and components

LED.E curing light (dentistry) is mainly composed by high power LED, optical fiber, main unit and charge device.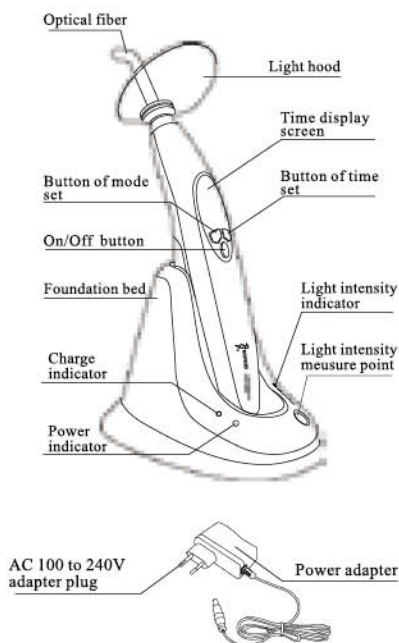

3. Technical specifications

- 1) Power supply:
 - a) Rechargeable Lithium battery, battery voltage and capacity: 3.7V, 1400mAh, battery model: ICR18490
 - b) Adapter input: AC100 ~ 240V 50/60Hz, Output: DC5V 1A, build-in fuse: 1A/250V.
- 2) Light source:
 - a) 5W high power blue light LED;
 - b) Wave length: 420~480nm;
 - c) Light intensity: 850~1000mW/cm².
- 3) Work condition:
 - a) Environment temperature: 5~40℃;
 - b) Relative humidity: ≤80%;
- 4) Size: 40×50×260mm.
- 5) Net weight: 147g.
- 6) Power consume: ≤8W.
- 7) Protection type against electrical shock: class II.
- 8) Protection against electrical shock: type B.
- 9) Protection against nocuousness: Ordinary equipment (IPX0).
- 10) Safety in the presence of flammable anesthetic mixture with air, oxygen or nitrous oxide: not suitable under this condition.

4. Installation and disassembly

- 1) Take off the red cap from the optical fiber, then insert the metal part into the front of main unit (make sure to screw when installing, oblique insert should be prevented).
- 2) Fix the light hood on the bottom of the optical fiber.
- 3) Uninstall the LED, just reverse the procedure above.
- 4) Battery replacement method: open the battery cover of the main unit, take the battery out, then disconnect the plug slightly. Connect the

plug of the new battery correctly, put the new battery in, and then fix the battery cover.

- 5) After using the machine or it is in need of charging, make the main unit head on inserted in the pedestal and compress it to make sure the main unit and pedestal chucking. When there is no need of charging, please pull out the pedestal power supply or put main unit into the pedestal inversely.

5. Operation

- 1) Press the mode button to set the working mode, the corresponding indicator will on when a mode set.
 - a) Full-power: the blue light radiates in full power.
 - b) Ramping: the blue light power increases stronger continually, after 5 seconds reaches to the highest power.
 - c) Pulse: the blue light works on the pulse condition.
- 2) Press the time button to set the solidifying time, 4 working time are available: 5, 10, 15, 20 seconds.
- 3) When operating, aim the optical fiber at the correct position, press the power button to start or stop emit of blue light.
- 4) During operation, the blue light can be stopped by press the power button at any time.
- 5) A battery detect circuit is fixed inside the main unit, when low power is detected, the indicator of the main unit winks, please charge in time.
- 6) Connect the power supply and the charge device correctly, put the main unit into the charge device, the yellow indicator will enlightened, when charging finish, the green indicator will enlightened and the yellow one will off. The green light is the power indicator of the charge device when not charge or charging finish. If an battery error is detected, the indicator of the charge device will be wink.
- 7) When operating finish, please clean the optical fiber with calico to avoid infecting the light intensity.
- 8) This equipment will turn off automatically if don't any action within 2 minutes, turn it on by press any button.
- 9) This equipment has the function of light intensity measuring. Connect the power supply of the light intensity measure device correctly, aim the top of optical fiber at the sensor, turn on the main unit to emit blue light, the green indicator of the light intensity measure device will indicate the value of the light intensity.
- 10) The effective light intensity of this equipment is much more higher than Halogen Lamp, The solidified depth of the curing light composites resin for 10 seconds will not less than 4mm.
- 11) The optical fiber can be sterilized in the condition of 135°C and 0.22M Pa.

6. Cautions

- 1) During operation, the light should be aimed straightly at the resin to ensure the effect of solidification.
- 2) Patients with biological reactions are forbidden to use this equipment.
- 3) Avoid aiming the blue light at eyes directly.
- 4) Please use the adapter and Lithium battery made or designed by Woodpecker. It will cause the potential risks to the operator or the patient when use others.
- 5) It is forbidden to touch the charging connector with metal or other conductor, to avoiding damage the circuit of charge and the battery.
- 6) Charging the battery in the condition of cool and ventilated. Please make sure of pressing out the buckle between the main unit and the pedestal, otherwise the battery charging might be failed because of the poor contact.

- 7) Do not disassemble the Lithium battery, it will lead to the circuit short or the electrolyte leakage.
- 8) Do not squeeze, shake and short the battery, do not store the battery with metal material.

7. Contraindication

- 1) Patients with biological reactions are forbidden to use this equipment.
- 2) The patients or doctors with heart pacemaker are forbidden to use this equipment.
- 3) The heart disease patient, pregnant woman and children should be cautious to use the equipment.

8. Daily maintenance

- 1) Only the optical fiber of this equipment can be autoclaved under high temperature and high pressure, other parts should be cleaned by clean water or neutral sterilized liquid, but do not soak the equipment in the water. Do not clean by volatile or soluble liquid, otherwise the marks on the control panel will fade.
- 2) Please clean the optical fiber to avoid the remaining resin on the surface and infect the life-span and the effectiveness of solidification.

9. Trouble shooting

Faulty	Possible cause	Solution
No indication No response	1. battery is out of power 2. faulty of battery 3. Battery is protected	1. Charge 2. change a new battery 2. change a new battery
The screen shows "Er"	Faulty of main unit	Repair
The power indicator winks	Low voltage	Charge, if it reappears after 15 minutes then change a new battery.
Light intensity is weak	1. The optical fiber is not installed correctly 2. there is crevice on the optical fiber 3. there is resin on the top of the optical fiber	1. Reinstall the optical fiber 2. change a new optical fiber 3. clean the resin
The equipment doesn't charge when the adapter is connected	1. The adapter is not connected well 2. faulty of adapter	1. Reconnect 2. change the adapter
Effective duration of the battery become short	The capacity of the battery decreased	Change a new battery
The mode indicator twinkles when charging	1. Low voltage 2. short-circuit of the battery	1. Back to normal after 15 minutes charging 2. change a new battery

If such handlings are completed, the equipment still cannot work normally, please contact with the special maintenance shop or our company.

10. Packing list

(The components of the equipment are listed in the packing list)

11. After-sales services

- 1) We offer two year warranty from the data of the equipment to be sold.
- 2) We won't responsible for any damage caused by non-professional person.

12. Storage and transportation

- 1) This equipment should be handled carefully, kept away from shaking point, installed or stored at shadowy, dry, cool and ventilated places.
- 2) Don't store the equipment together with articles that are combustible, poisonous, caustic, and explosive.
- 3) This equipment should be stored in humidity is $\leq 80\%$, the atmosphere pressure is $75 \sim 106 \text{ kPa}$ and the temperature is $-10^{\circ}\text{C} \sim +55^{\circ}\text{C}$.
- 4) Excess impact or shake should be prevented during transportation, lay it carefully and do not invert it.
- 5) Don't put it together with dangerous articles during transportation.
- 6) Don't solarize, don't get wet in rain or snow during transportation.

13. Symbol instructions

Trademark

Type B applied part

IPX0

Ordinary equipment

Caution, consult accompanying documents

Class II equipment

Use indoor only

CE marked product

FDA marked product

Date of manufacture

Manufacturer

Screw inside/outside

Authorised Representative in the EUROPEAN COMMUNITY

14. For technical data, please contact

Wellkang Ltd (www.CE-marking.eu)
29 Harley St., London W1G 9QR, UK

15. Environmental protection

No harmful elements in this equipment, it can be disposed according to the local laws.

GUILIN WOODPECKER MEDICAL INSTRUMENT CO., LTD

Registration Add and Manufactory Add:

Information Park, National High Tech Zone. Guilin, Guangxi, P. R. China 541004

carlo de giorgi s.r.l.

Via Tonale, 1 – 20021 Baranzate (MI) – Italy

Tel. +39.02.356.15.43 r.a. – Fax +39.02.356.18.08