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CONTENTS

1 - PRINCIPLE AND USAGE	3
2 - STRUCTURE AND COMPONENTS	3
3 - TECHNICAL SPECIFICATIONS.....	3
4 - OPERATION WAY.....	4
5 - INSTRUCTION OF RECHARGING.....	4
6 - CAUTIONS	6
7 - DAILY MAINTENANCE	6
8 - EXCLUSION OF PROBLEMS	7
9 - PACKING LIST.....	8
10 - AFTER-SERVICE	8
11 - STORAGE AND TRANSPORTATION	8
12 - MARK ILLUSTRATION	9
13 - ENVIRONMENTAL PROTECTION	9
14 - FOR TECHNICAL DATA PLEASE CONTACT.....	9

1 - PRINCIPLE AND USAGE

1. LED-B adopts the principle of ray radiation to solidify the light-sensitive resin by shooting at it in a short time.
2. This product is used to restore teeth and solidify material for whitening teeth.

2 - STRUCTURE AND COMPONENTS

LED-B(dental) is composed mainly of high power LED, fiber optic and main unit.

3 - TECHNICAL SPECIFICATIONS

1. Power supply: rechargeable Lithium accumulator
Accumulator model: CMICR18650F
Accumulator voltage and capacity: 3.7V/2200mAh
Input of charger: ~100V-240V 50Hz/60Hz
Build-in fuse of charger: 1A/250V
2. Light source:
 1. blue light
 2. wave length: 420nm—480nm
3. Working conditions:
 1. environment temperature: 5°C – 40°C
 2. relative humidity: ≤ 80%
4. Dimension: 31 mm × 34 mm × 260 mm
5. Net weight: 145 g
6. Consumption power: ≤8 W
7. Protection type against electrical shock: class
8. Protection against electrical shock: type B
9. Protection against harmful ingress of water or particular matter: ordinary equipment (IPXO)
10. Safety in the presence of flammable anesthetic mixture with air, oxygen or nitrous oxide: not suitable under this condition.

4 - OPERATION WAY

1. Take the red protection cap off the fiber optic and then insert metal cap of the fiber optic into the LED-B's installation hole. (You must insert the parts to the end point by circumgyrative way , be sure not sloping insert.)
2. To install the light hood on the top of the fiber optic .
3. Press the time button to choose the solidification time. 4 working time modes are available: 5, 10, 15, 20 seconds.
4. During the operation, aim the light at the position needing solidification, press the on/off button and the handpiece gives off a "di" sound. At the same time, weak blue light comes out. Three seconds later comes the strong light and the digital screen begins to count down until 0, which means that the operation is over.
5. After operation, please clean the fiber optic with cotton cloth if there is resin remained on it to ensure the light power next time.
6. The depth of solidification of composite is no less than 3mm per 10 seconds.
7. The fiber optic can be spinned off by 360° and autoclaved under high temperature of 135 °C and pressure of 0.22MPa.
8. During operation, if the indicator light of capacity is on, it means low volume. Recharge it at once.

5 - INSTRUCTION OF RECHARGING

1. Recharging by UDS-M scaler: press the recharging button and the indicator light shines faintly. Insert the Led-B into the recharging hole and the light becomes strong, which means that the recharging is on. When the light twinkles, it means the recharging is over.
2. Recharging by pedestal: connect the charger with the pedestal and get through the electricity. Put the LED-B into the pedestal and then the indicator light is on. When the light begins to twinkle, it means the recharging is over.
3. The accumulator has no memory and can be recharged any time.
4. The first recharging time must last over 10 hours to ensure adequate power for operation. At other times, 4-6 hours is enough.

6 - CAUTIONS

1. During operation, the light should be aimed straight at the resin to ensure solidification effect.
2. Patients with biological reactions are forbidden to be used with.
3. It is forbidden to shoot the blue light at eyes.
4. It is advised to use the original pedestal and charger because other brand pedestal and charger are likely to damage the Lithium accumulator and the circuit.
5. It is forbidden to touch the charging needle of the pedestal or the contact area of the LED-B with metal or other conductors. In order not to damage the circuit board and the Lithium accumulator short-circuit.

7 - DAILY MAINTENANCE

1. Only the fiber optic can be autoclaved under high temperature and pressure. As to other fittings, wipe the surface with clean water or distilling liquid. No immersing.
2. After every operation, shut off the electricity and check the fiber optic whether there is resin remained on it. Wipe off if any.

8 - EXCLUSION OF PROBLEMS

Problems	Possible causes	Solutions
Inadequate Light power	1. The fiber optic is not Inserted till the bottom. 2. There is leakage in the fiber optic. 3. There is resin remained on the fiber optic. 4. The accumulator's capacity is inadequate.	1. Insert again correctly 2. Change the fiber optic. 3. Wipe off the resin. 4. Charge the LED-B
The LED-B doesn't Work	1. The accumulator's capacity is used up. 2. The LED-B works continuously too long and the heat protection system works. 3. LED is damaged.	1. Charge the LED-B 2. Stop the operation for several minutes. 3. Contact the local distributor or us.

9 - PACKING LIST

- Main unit
- Fiber optic
- Light hood
- Charger
- Pedestal
- Instruction manual

10 - AFTER-SERVICE

1. We offer one year free repair to those products that can't work because of quality problems, counting from the sales date.
2. Those irrevocable damages caused by non-professionals don't fall into this sphere.

11 - STORAGE AND TRANSPORTATION

1. This device should be handled carefully, kept away from shaking point, installed or stored at shadowy, dry, cool and ventilated places.
2. Don't store it together with articles that are combustible, poisonous, caustic and explosive.
3. This device should be stored in the environment where the relative humidity is $\leq 80\%$, the atmosphere pressure is 75kPa~106kPa and the temperature is $-10^{\circ}\text{C} \sim 55^{\circ}\text{C}$.
4. Excess impact or shake should be avoided during transportation.
5. Don't mix it with dangerous articles during transportation.
6. Keep it from sun or snow or rain during transportation.

12 - MARK ILLUSTRATION

Trademark

Class II Equipment

Type B equipment

Caution ! Please read the instruction

Used indoor only

IPX0

Ordinary equipment

CE marked product

13 – ENVIRONMENTAL PROTECTION

There is not any harmful element in this device. It can be disposed of according to the local laws.

We reserve the rights to alter the design, the technology and fitting parts, instruction manual and packaging without further notice. If there is any discrepancy between the picture and the real product, please refer to the real product.

14 - FOR TECHNICAL DATA PLEASE CONTACT

Carlo De Giorgi s.r.l.
Via Tonale 1
20021 Baranzate
Milano ITALY
Tel. +39 02 3561543
Fax +39 02 3561808
<http://www.degiorgi.it>

carlo de giorgi s.r.l.

Via Tonale, 1 – 20021 Baranzate (MI) – Italy
Tel. +39.02.356.15.43 r.a. – Fax +39.02.356.18.08
<http://www.degiorgi.it> E-Mail: info@degiorgi.it