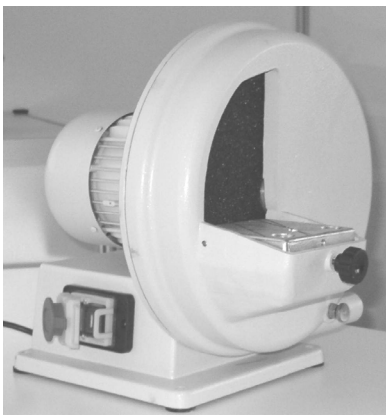
Manuale di Istruzione Uso e Manutenzione
User and Maintenance manual
Manuel d'Utilisation et d'Entretien
Gebrauchs- und Wartungsanleitung
Manual de Instrucciones de Uso y Mantenimiento

Squadramodelli
Dental model trimmer
Rodoirs
Modellschleifgerät
Escuadramodelos

carlo de giorgi s.r.l.

Dati modello

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CONTENTS

1 - STANDARDS AND GENERAL PRECAUTIONS	4
1.1 - INTRODUCTION	4
1.2 - DESCRIPTION OF THE APPLIANCE	4
1.3 - REFERENCE STANDARDS	4
1.3.1 - Obligatory compliance	4
1.3.2 - Voluntary compliance	4
1.4 - RESPONSIBILITIES OF THE CUSTOMER.....	4
1.5 - REQUESTS FOR SERVICE AND SPARE PARTS.....	5
2 - TECHNICAL FEATURES	5
2.1 - TECHNICAL FEATURES	5
3 - INSTALLATION	5
3.1 - POSITIONING	5
3.2 - CONNECTIONS AND PLACING INTO SERVICE	5
4 - OPERATION AND USE	6
4.1 - OPERATION	6
4.2 - OPERATOR	7
4.3 - INTENDED USE.....	7
4.4 - SAFETY WARNINGS.....	7
5 - OPERATOR INSTRUCTIONS	8
5.1 - CONTROLS.....	8
5.2 - INSTRUCTIONS FOR USE	9
5.2.1 – Stopping and emergency stop modes	9
5.2.2 - Warning	9
6 - MAINTENANCE	9
6.1 – Cleaning and lubrication	9
6.1.1 – Changing the grinding disk	9
7 - DIAGNOSTICS.....	10
8 - ACCESSORIES.....	10
8.1 – ORTHODONTICS KIT	10
8.2 – FINE GRINDING DISK	10

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declares, under its own responsibility, that the apparatus:

Type: **DENTAL MODEL TRIMMER**

described as:

appliance designed for trimming objects made of plaster, utilized in dental laboratories.

conforms to the Laws implementing the requirements of the "Machinery Directive" 2006/42/CE, the "Low Voltage Directive" 2006/95/CE and the "Electromagnetic Compatibility" Directive 2004/108/CE.

Signed:

De Giorgi Ariberto
Managing Director

1 - STANDARDS AND GENERAL PRECAUTIONS

1.1 - INTRODUCTION

This manual is the property of CARLO DE GIORGI S.r.l. The contents of this manual cannot be reproduced or disclosed to other parties. All rights reserved.

1.2 - DESCRIPTION OF THE APPLIANCE

The appliance in question is designed to trim plaster models in dental laboratories; the tool comprises an abrasive disk in the case of model TRIM-1, and a diamond disk for model TRIM ECO DISK.

1.3 - REFERENCE STANDARDS

1.3.1. - Obligatory compliance

- Directive 2006/42/CE – Machinery Directive
- Directive 2006/95/CE – Low voltage Directive (Italian leg.: Law n° 791/1977, 626/1996, 277/97)
- Directive 2004/108/CE relative to Electromagnetic Compatibility (EMC) - (Italian legislation: Decree Law 6 November n. 194/2007)
- CEE directive n. 93/68 – Amendment of directives 73/23/CEE and 89/336/CEE relative to CE marking (Italian legislation n° 626/1996, and n° 277/97)
- DPR 27.4.1955 n.547, "Work accident prevention regulations"
- DPR 27.4.1956 n.303, "General work hygiene regulations"
- Italian Decree Law 12 August 1991 n.277 implementing EEC directives n.80/605; n.83/477; 86/188; 88/642 concerning the protection of workers against risks deriving from exposure to chemical, physical and biological agents in the workplace, as defined by art. 7 of Law 30 July 1990, n.212

1.3.2 - Voluntary compliance

- EN ISO 12100 (2005) Machine safety – Basic concepts; general design principles – Part 1 – Basic terminology and methods (ex EN 292-1) /Part 2 – Specifications and technical principles (ex EN 292-2).
- EN 60204-1 Machine safety – Machine electrical equipment - Part 1: General requisites.

1.4 - RESPONSIBILITIES OF THE CUSTOMER

The user must install the dental model trimmer in a suitable place equipped with an electrical system that complies with established legislation.

The appliance must be installed in a dry environment that is illuminated in accordance with the requirements of established legislation.

NOTE: The expression "established legislation" refers to the laws in force in the user's country.

1.5 - REQUESTS FOR SERVICE AND SPARE PARTS

For all electrical maintenance requirements, contact CARLO DE GIORGI di Baranzate di Bollate, Via Tonale, 1, Milan, Italy.

In the case of faults, notify CARLO DE GIORGI technical personnel of the exact nature of the problem, specifying also the lot number shown on the label affixed to the rear of the appliance.

2 - TECHNICAL FEATURES

2.1 - TECHNICAL FEATURES

		TRIM - 1	TRIM ECO DISK
Voltage	V	230/120	230/120
Frequency	Hz	50/60	50/60
Installed power	W	500	500
Max. tool speed	rpm	1,400	1,400
Mass	kg	14	14
Noise level in no-load condition	dB	60	60

3 - INSTALLATION

3.1 - POSITIONING

The appliance is supplied assembled and ready for use after connecting the power cable, the grinding dust suction system and, in the case of TRIM-1 models, the water supply hose to an external tap.

The package contains:

- a) appliance;
- b) aluminium plate;
- c) rubber hose for draining residual water (model TRIM-1 only);
- d) rubber hose for water inlet connection (model TRIM-1 only);
- e) 4 and 6 mm Allen keys;
- f) plate locking knob
- g) user instructions.

The appliance can be moved by hand without requiring lifting devices.

3.2 - CONNECTIONS AND PLACING INTO SERVICE

After the appliance has been positioned, connect the power cable by inserting the plug into a mains socket outlet with overcurrent protection devices.

4 - OPERATION AND USE

4.1 - OPERATION

The appliance should be placed on a work table in front of the operator, at a convenient height to reduce fatigue and unnecessary effort.

Secure the appliance to the table by means of four M6 bolts inserted through the holes in the baseplate and secured under the worktable by means of nuts.

Before setting the appliance into operation:

- a) connect the water supply hose to the threaded connector (1, Fig. 1) located on the rear of the appliance - TRIM-1 models only;
- b) the residual water drain hose on the relevant connector (2, Fig. 1) - TRIM-1 models only - inserting the other end of the hose into a decanting tank; or
- c) a suitable suction system (we recommend the AIR-BOX DE GIORGI system) on the relevant connection (2, Fig. 1) - TRIM ECO DISK models only;

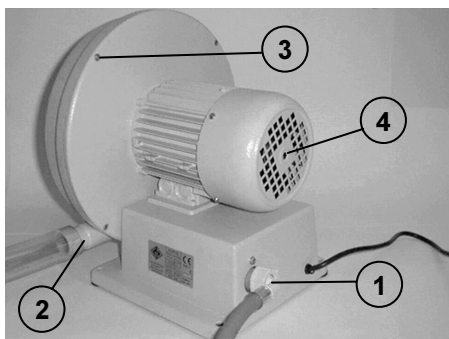
Fig. 1

The appliance is set into operation by pressing pushbutton “I” (5, Fig. 2a).

Remember to switch the appliance OFF after use by pressing pushbutton “O” (6, Fig. 2a) through the mushroom-head stop button (11, Fig. 2b). Lock the mushroom-head stop button in the closed position (Fig. 3a)

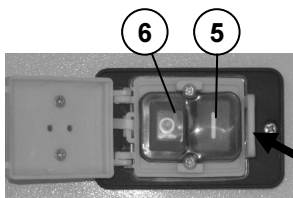
Fig. 2a

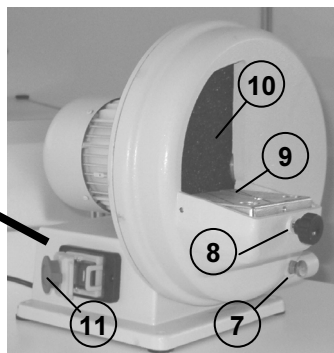
Fig. 2b

4.2 - OPERATOR

The operator starts the appliance by means of a voluntary command thereby assuming direct control of its operation.

4.3 - INTENDED USE

The appliance is designed for use in dental laboratories. The plate (9, Fig. 2b) is used as a rest for the workpiece and may be inclined in three positions ($90^\circ \pm 10^\circ$) thanks to the presence of three alternative holes in which to tighten the positioning knob (8, Fig. 2b).

The plate features a series of graduated lines that the operator can follow when tilting the workpiece in the horizontal plane.

The appliance must not be used for any purposes other than those established by the manufacturer.

4.4 - SAFETY WARNINGS

- The appliance must be used exclusively by skilled personnel who are aware of the potential hazards;
- do not use the appliance in the vicinity of inflammable materials;
- do not open or disassemble the appliance when it is running;
- do not subject the appliance to excess workloads to prevent rapid deterioration of the grinding disc, degradation of performance levels or overheating of the motor;
- do not wear clothing with loose fitting sleeves, scarves, chains or necklaces, bracelets, etc. which could become entrapped by moving parts of the appliance;
- always wear safety glasses and, in the event of prolonged use, suitable hearing protection devices (hearing defenders, earplugs, etc.);

- when working with model TRIM-1 it is obligatory to drain residual water in a decanting tank and also to keep an updated register for the disposal of waste sludge;
- when working with model TRIM ECO DISK you must use a suction system to exhaust machining dust by making a connection to the appliance dust outlet (2, Fig. 1). We recommend the use of the AIR-BOX DE GIORGI suction system.

5 - OPERATOR INSTRUCTIONS

5.1 - CONTROLS

The dental model trimmers are equipped with the following control devices located on the front:

- a) Green start pushbutton (5, Fig. 2a)
- b) Red stop pushbutton (6, Fig. 2a)
- c) Red mushroom-head stop button (11, Fig. 2b)

To switch the machine on using the start pushbutton, (5, Fig. 2a), the mushroom-head stop button must be opened (11, Fig. 2b), unlocking it from its closed position (Fig. 3a), by pushing it in the direction indicated by the arrow on the mushroom-head button itself.

Fig. 3a

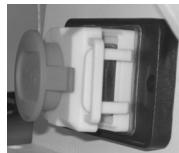
Fig. 3b

The appliance is equipped with a power circuit disconnect device which trips in the event of electrical power loss.

Setting the appliance into operation automatically starts the flow of water from the inlet connection (1, Fig. 1); the flow rate can be adjusted by means of the valve on the front of the appliance (7, Fig. 2b) (model TRIM-1 only).

When the appliance is operating, check that the mushroom-head button device is positioned ajar (Fig. 3b), to ensure rapid intervention in the case of emergency.

5.2 - INSTRUCTIONS FOR USE

5.2.1 – Stopping and emergency stop modes

The appliance can be stopped as follows:

- pushing the mushroom-head stop button (11, Fig. 2b), which completely interrupts power supply to the appliance;
- by unplugging the power cable from the mains socket outlet.

5.2.2 - Warning

DO NOT ABANDON THE APPLIANCE WHILE IT IS RUNNING AND REMEMBER TO SWITCH IT OFF WHEN YOU FINISH WORKING WITH IT. IF YOU NOTICE MALFUNCTIONS STOP THE APPLIANCE IMMEDIATELY AND DISCONNECT IT FROM THE ELECTRICAL POWER SUPPLY. BEFORE RECONNECTING THE POWER SUPPLY FIND THE CAUSE OF THE FAULT AND REMEDY IT. IN THE CASE OF PROBLEMS, CONSULT CARLO DE GIORGI.

6 - MAINTENANCE

Any maintenance work on the electrical parts inside the appliance casing must be performed exclusively by authorized and suitably skilled personnel. Note that the appliance carries dangerously high voltage which represents a hazard if maintenance is performed on the electrical components without first disconnecting the power supply.

6.1 – Cleaning and lubrication

The appliance does not require particular maintenance with the exception of regular cleaning of the working area: before cleaning the appliance always disconnect it from the power supply.

6.1.1 – Changing the grinding disk

To replace the grinding disk proceed as follows:

- a) disconnect the appliance from the electrical power supply;
- b) remove the disk guard by unscrewing the four screws at the rear (3, Fig. 1) using the 4 mm Allen key supplied with the appliance;
- c) immobilize the motor shaft at the rear (4, Fig. 1) using the 6 mm Allen key supplied with the appliance, and then unscrew the screw at the front of the appliance using an M 13 wrench;
- d) remove the worn grinding disk (10, Fig. 2b) (the grinding disk can also be inverted so that the rear face can be used), fit the new disk or invert the original one and tighten the screw at the front while holding the motor shaft with the Allen key; refit the disk guard.

7 - DIAGNOSTICS

If malfunctions and faulty operation persist, set the main power switch to OFF, unplug the appliance and consult CARLO DE GIORGI.

WARNING!

ALL WORK ON THE APPLIANCE MUST BE PERFORMED WITH THE POWER SUPPLY DISCONNECTED

8 - ACCESSORIES

8.1 – ORTHODONTICS KIT

The plate and relative accessories in the kit (11, Fig. 4) are designed to allow model trimming in a vertical and horizontal direction.

8.2 – FINE GRINDING DISK

Fig. 4 shows a diamond disk (12) and fine disk (13)

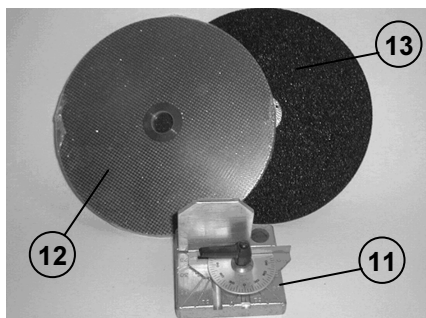
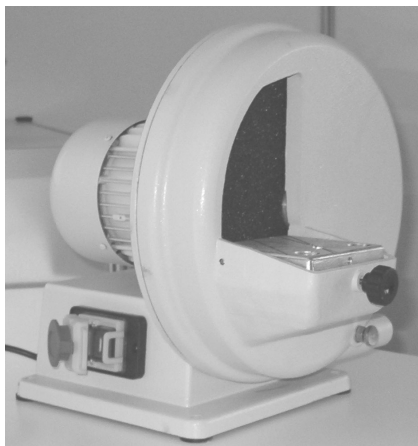
Fig. 4

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